Brugsanvisning

Unitron Medical Cabinet 135 liter

Unitron a/s

DK

Brugsanvisning

Specifikationer

135 liter / 240 liter

Dimensioner

HxBxD 85x55x56 cm
140x54x55 cm

Kapacitet 135 liter / 240 liter

Vægt.....35 kg / 50 kg

Strømforsyning

..... 230 VAC 50Hz 85W

Omgivelses-

temperatur 0-35°C

Justerbar

temperaturområde 1-7°C

Forprogrammeret

temperatur 5°C

Temperaturudsving 1°C

Kølemiddel R600

Før opstart

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Item: 1013-1085

Unpacking

Make sure that the Medicine Cabinet is intact. Read Manual.

Installation

Install in a dust free environment. Keep a 1" clearance at the sides/top and 3" at the back of the Medicine Cabinet to allow free circulation of air. In order to conserve electricity do not put your Medicine Cabinet near a source of heat, such as a cooker or radiator.

If cabinets are delivered on its side, leave it standing upright for minimum 2 hours before plugged in.

After plugging in the power supply check that the fan inside the Medicine Cabinet is activated. The fan starts at the same time as the compressor. This ensures the best temperature stability.

Instructions

The pre-programmed temperature is set to 5°C. The display temperature is the air-temperature inside the cabinet. There is a $\pm 1^{\circ}\text{C}$ deviation from this setting.

Note that the temperature shown on the display is the air temperature inside the cabinet

Maintenance

The Medicine Cabinet can be cleaned with a moist cloth. Other detergents than dishwashing liquid might damage the surface. Dust can be removed with gentle air pressure or a soft brush.

Service and warranty

There is a 12 months warranty on the Medicine Cabinet covering in parts or production. Damage caused by incorrect use is not covered by the warranty nor is the manufacturer liable for compensation due to malfunction.

www.unitron.dk

Unitron a/s · Kokmose 6 · DK-6000 Kolding · Tlf. +45 75802122 · mail@unitron.dk

Work smarter – not harder