

# Unitron a/s



# PiggyTrim

## - modern hoof care

Hoof trimming crate for sows – simple and gentle hoof care.

Regular hoof trimming is important for both animal welfare and production. When a sow stands and walks effortlessly, it will both eat and drink better promoting a high milk potential. A good hoof trimming crate makes the job easier, faster and safer – for both the sow and manual worker.

The PiggyTrim Hoof Care crate is built for systematic hoof trimming in sow herds. The sow walks in on its own, and the crate is then lifted hydraulically. When the sow is lifted like this it settles down and stays calm, so the work can be done at a comfortable height with free access to all four hooves.

The crate can be placed wherever it fits best in the barn. It's available with fittings for both solid and slatted floors. When lifted, there's plenty of space underneath for free passage, so it doesn't get in the way – even in narrow passages.

### Hoof trimming in practice

Trimming usually takes around 5–10 minutes per sow and can be done with a one-hand angle grinder. Once hoof care becomes a regular part of herd management, the number of lame sows drops quickly. Healthy hooves make it easier for the sow to get up, move effortlessly and nurse the piglets – which improves both welfare and productivity.

Scan the code to watch videos



- **Gentle and calm for the sow**
- **Comfortable working height**
- **High ground clearance**
  - can be placed in walkways
- **Simple two-button control**
- **Easy access to all hooves from both sides**



Item: 1116-1020



### Piggy Lanolin Spray 250 ml 12 pcs. packages

A natural product based on lanolin which makes it extremely adhesive and thus very protective.

Item: 1116-1804



### Piggy Salicylic Care Paste 300 ml

Piggy Salicylic Care Paste contains 38% salicylic acid and is used for hoof trimming as disinfection under cotton wool and bandage. Medium thick gel that sticks easily to the hoof and cotton wool.

The effective adhesion and disinfection also make it suitable for the treatment of shoulder wounds under the shoulder cap for sows.

Item: 1223-0233



### Piggy Care Spray 300 ml 12 pcs. packages

Skin repair & protection. Durable, quick-drying and protective spray to support skin healing.

Item: 1116-1805



### Piggy Tar spray 400 ml 12 pcs. packages

Tar has been a well-known and good hoof product for many years, as the natural antiseptic properties of tar are used.

Item: 1116-1803



### Piggy Dry Pads 12 pcs.

Specially shaped non woven pads for better treatment of interdigital/digital dermatitis. Can be used with Piggy Salicylic Care Paste. The pad is designed to fit in between the two claws before wrapping.

Item: 1116-1689



### Piggy Dry Pads 50 pcs.

Specially shaped non woven pads for better treatment of interdigital/digital dermatitis. Can be used with Piggy Salicylic Care Paste. The pad is designed to fit in between the two claws before wrapping.

Item: 1116-1691



### Piggy Pro bandage 9 cm x 6 meters

Blue. 12 pcs. packages

Item: 1116-1681



### Piggy Pro bandage 9 cm x 6 meters

Red. 12 pcs. packages


Item: 1116-1682



### Piggy hoof disc True grit

- The disc are coated with heavy stone
- Suitable for one-handed angle grinder
- Quick pruning and long life
- Can be used on wet and dry hooves
- Dia.: 115 mm with 22.23 mm hole
- No dust or smell

Item: 1116-1207

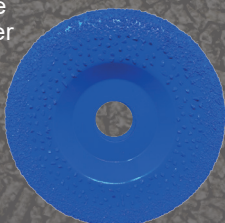

### Metal gloves for hoof care

Can be used on both hands.

Including fabric glove.

Size Medium:

Item: 1116-7301

Size XL:

Item: 1116-7302



### Piggy Care Starter Kit for bandaging - 50 Sows

Treatment package for 50 x wounds on legs and hooves.

- Piggy Pro bandage
- Piggy Dry Pads
- Piggy Salicylic Care paste
- Piggy Lanolin Spray
- Piggy Tar spray
- Piggy Care spray
- Piggy Pistol
- Piggy Bandage knife

Item: 1116-1692



### Piggy Care Starter Kit for hoof care

Indeholder: Piggy Pro bandage, Piggy Dry Pads, Piggy Salicylic Care paste, Piggy Lanolin Spray, Piggy Tar spray, Piggy Care spray, Piggy hoof disc true grit, Piggy Pistol, Piggy bandage knife, Demotec hoof check, Metal gloves, Angle grinder, Hearing protection, Dust mask, Safety glasses

Item: 1116-1693

